




J DENTAL CARE

just smile

JD EVOLUTION S

Safe as always, thinner than ever Ø 3.2mm

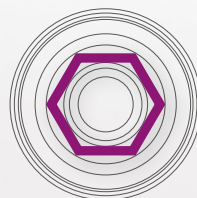


hexagonal connection



Made in Italy

SAFE AS ALWAYS, THINNER THAN EVER

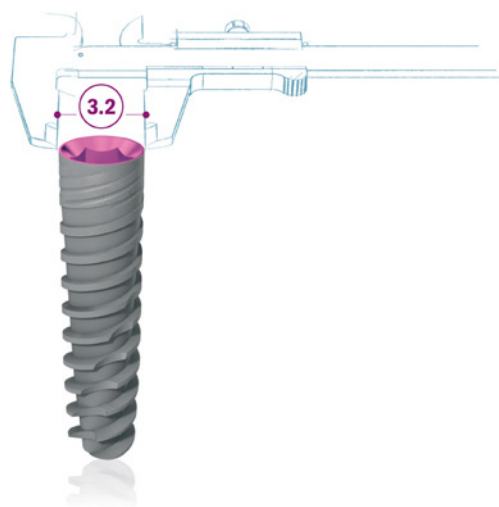

advantages


- High primary stability
- Self-cutting thread design
- The flat cutting apex allows for final redirection of the implant during its placement
- Immediate function capability
- Built-in Platform switching
- Solves problems of tight spaces
- Designed specifically for narrow spaces in the esthetic zone

IMPLANT MEASURES AND LENGTHS

The JDEvolution S dental implants are available in the lengths shown in the following table:

IMPLANT DIAMETER	LENGTH	TIP DIAMETER	PLATFORM DIAMETER	ABUTMENT INTERFACE
Ø 3,2	8, 10, 11.5, 13, 15	2.8	3.2	3.0

Note: all measurements in mm

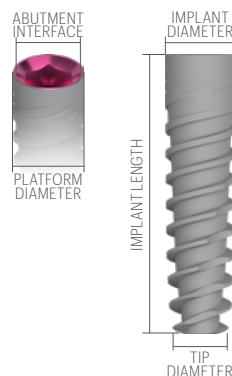


DRILLING SEQUENCE

It is recommended to adhere to the indications of the following drilling sequence to ensure optimal primary stability of the implants

IMPLANT DIAMETER	SOFT BONE TYPE IV	MEDIUM BONE TYPE II-III	DENSE BONE TYPE I
Ø 3,2	1,5 (2)	2 2,4	2 2,4 (2,8)

Note: all measurements in mm

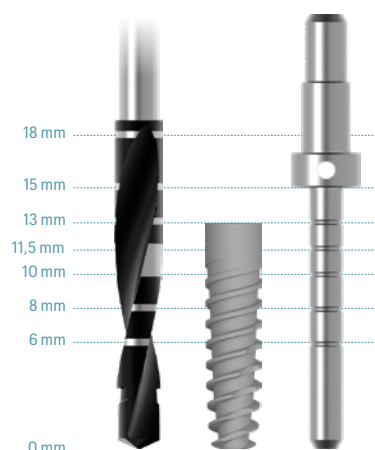


PRODUCT CATALOGUE

Implants:

Ø 3,2

EV32080	JDEvolution S Ø 3.2 L 8
EV32100	JDEvolution S Ø 3.2 L 10
EV32115	JDEvolution S Ø 3.2 L 11.5
EV32130	JDEvolution S Ø 3.2 L 13
EV32150	JDEvolution S Ø 3.2 L 15

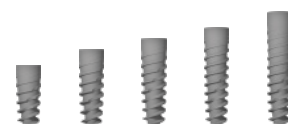

Cover Screw:

ESCS	Cover Screw JDEvolution S
------	---------------------------



Healing Abutment:

ESHA323	Healing Abutment Ø 3.2 H 3.0 JDEvolution S
ESHA325	Healing Abutment Ø 3.2 H 5.0 JDEvolution S
ESHA327	Healing Abutment Ø 3.2 H 7.0 JDEvolution S
ESHA403	Healing Abutment Ø 4.0 H 3.0 JDEvolution S
ESHA405	Healing Abutment Ø 4.0 H 5.0 JDEvolution S
ESHA407	Healing Abutment Ø 4.0 H 7.0 JDEvolution S




Impression Coping:

OPEN TRAY

ESICOT32C	Impression Coping Open Tray Ø 3.2 JDEvolution S
ESICOT40C	Impression Coping Open Tray Ø 4.0 JDEvolution S
ESICOTS	Impression Coping Open Tray Screw JDEvolution S

CLOSED TRAY

ESICCT32C	Impression Coping Closed Tray Ø 3.2 JDEvolution S
ESICCT40C	Impression Coping Closed Tray Ø 4.0 JDEvolution S
ESICCTS	Impression Coping Closed Tray Screw JDEvolution S

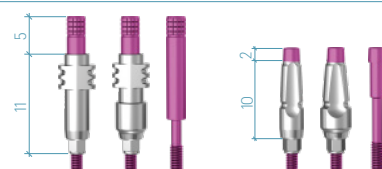

Temporary Abutment:

ESTAEC	Temporary Abutment Engaging JDEvolution S
ESTANEC	Temporary Abutment Non Engaging JDEvolution S

Torque recommended 30 Ncm



Immediate Temporary Abutment:

ESIT5C	Immediate Temporary Abutment JDEvolution S
EVITC	Immediate Temporary Abutment Healing Cap



GP Abutment:

ESGPAE32C	GP Abutment Ø 3.2 JDEvolution S
ESGPAE40C	GP Abutment Ø 4.0 JDEvolution S
ES15GPAE40C	GP Abutment 15° Ø 4.0 JDEvolution S
ES25GPAE40C	GP Abutment 25° Ø 4.0 JDEvolution S

Torque recommended 30 Ncm



Anatomic Abutment:

ESAA15C	Anatomic Abutment Ø 4.0 H 1.5 JDEvolution S
ESAA30C	Anatomic Abutment Ø 4.0 H 3.0 JDEvolution S

Torque recommended 30 Ncm



Wax up Abutment:

ESWAEC	Wax-Up Abutment Engaging JDEvolution S
ESWANEC	Wax-Up Abutment Non Engaging JDEvolution S

Torque recommended 30 Ncm



Cobalt-Chrome Abutment:

ESCCCEC	Cobalt-Chrome Abutment Engaging JDEvolution S
ESCCNEC	Cobalt-Chrome Abutment Non Engaging JDEvolution S
EVCACCNEC	Cobalt-Chrome Conical Abutment Non Engaging JDEvolution

Torque recommended 30 Ncm



ESS	Prosthetic Screw JDEvolution S	Torque recommended 30 Ncm	
ESSA	Prosthetic Screw Angulated JDEvolution S *		
	* for this screw use Angulated Screw Driver for JDTorque		

ESCA15C	Straight Conical Abutment H 1.5 JDEvolution S	Torque recommended 30 Ncm	
ESCA30C	Straight Conical Abutment H 3.0 JDEvolution S		
ESCA15TSC	Conical Abutment Tissue Straight H 1.5 JDEvolution S		
ESCA30TSC	Conical Abutment Tissue Straight H 3.0 JDEvolution S		
ESCA1725C	17° Angulated Conical Abutment H 2.5 JDEvolution S	Torque recommended 30 Ncm	
ESCA1735C	17° Angulated Conical Abutment H 3.5 JDEvolution		
ESCA3025C	30° Angulated Conical Abutment H 2.5 JDEvolution		
ESCA3035C	30° Angulated Conical Abutment H 3.5 JDEvolution S		
ESCA1725TSC	Conical Abutment Tissue Angulated 17° H 2.5 JDEvolution S	Torque recommended 30 Ncm	
ESCA1735TSC	Conical Abutment Tissue Angulated 17° H 3.5 JDEvolution S		
ESCA3025TSC	Conical Abutment Tissue Angulated 30° H 2.5 JDEvolution S		
ESCA3035TSC	Conical Abutment Tissue Angulated 30° H 3.5 JDEvolution S		
ESCASA	Angulated Conical Abutment Screw JDEvolution S	Torque recommended 30 Ncm	
EVCAPS	Prosthetic Screw Conical Abutment JDEvolution		
EVCAPSA	Prosthetic Screw Angulated for Conical Abutment JDEvolution *	Torque recommended 15 Ncm	
EVCAPSN:	Prosthetic Screw Conical Abutment Direct N **		
EVCAPSNA	Prosthetic Screw for Angulated Channel Conical Abutment Direct N *		
EVCAICOTEC	Conical Abutment Impression Coping Open Tray Engaging JDEvolution		
EVCAICOTC	Conical Abutment Impression Coping Open Tray JDEvolution		
EVCAICOTLC	Conical Abutment Impression Coping Open Tray Long JDEvolution		
EVCAICOT02	Screw for Conical Abutment Impression Coping Open Tray JDEvolution		
EVCAICOT04	Screw for Conical Abutment Impression Coping Open Tray Long JDEvolution		
EVCAICCTC	Conical Abutment Impression Coping Closed Tray		
EVAHC	Conical Abutment Healing Cap JDEvolution		
EVAHCL	Conical Abutment Healing Cap H 6.0 JDEvolution		
EVAHC9	Conical Abutment Healing Cap H 9.0 JDEvolution		
EVAHCB	Conical Abutment Healing Cap Peek JDEvolution	Torque recommended 15 Ncm	
EVCAGPAEC	GP Abutment Engaging for Conical Abutment JDEvolution		
EVCAGPANEC	GP Abutment Non Engaging for Conical Abutment JDEvolution		
EVCATANEWSC	Temporary Abutment Non Engaging Conical Abutment Smooth for Welding JDEvolution		
EVCATANEC	Temporary Abutment Non Engaging Conical Abutment JDEvolution		
EVCATANEWSC	Temporary Abutment Non Engaging Conical Abutment for Welding JDEvolution		
EVCWANEC	Wax-Up Abutment Non Engaging for Conical Abutment		
EVCAAR	Conical Abutment Replica		

* for this screw use Angulated Screw Driver for JD Torque

** for screw use JDEvolution Plus prosthetic screwdriver

ESCA15MIC	Conical Straight Abutment MINI H 1.5 JDEvolution S	
ESCA30MIC	Conical Straight Abutment MINI H 3.0 JDEvolution S	
EVCAHCMIC	Conical Abutment MINI Healing Cap	
EVCAHCLMIC	Conical Abutment MINI Healing Cap H 6.0	
EVCAITMIC	Interface CAD CAM Non Engaging H 7.0 for Conical Abutment MINI	
EVCAITMIEC	Interface CAD CAM Engaging H 7.0 for Conical Abutment MINI	
EVCATIBMIC	Interface CAD CAM Non Engaging H 5.0 for Conical Abutment MINI	
EVCATIBMIEC	Interface CAD CAM Engaging H 5.0 for Conical Abutment MINI	
EVCATANEMIC	Temporary Abutment Non Engaging Conical Abutment MINI	
EVCAICOTMIC	Conical Abutment MINI Impression Coping Open Tray	
EVCAICOTMIEC	Conical Abutment MINI Impression Coping Engaging Open Tray	
EVCAARMI	Conical Abutment MINI Replica	
EVCAARMIC	Conical Abutment MINI Replica CAD CAM	
EVCASBCEMIC	ScanBody Conical Abutment MINI	
EVCASBCEMIEC	ScanBody Engaging Conical Abutment MINI	
EVCASBCESMIC:	On Top ScanBody Conical Abutment MINI	

TiBase, JDScanBody and Implant Replica CAD CAM:

ESSBCEC	ScanBody Engaging JDEvolution S
EVCASBCEC	ScanBody Conical Abutment JDEvolution
EVCASBCEEC	ScanBody Engaging Conical Abutment
EVCASBCECSC:	On Top ScanBody Conical Abutment JDEvolution Plus **
JDSCGIOSE	ScanBody Full Arch Tracer Hexagonal **
JDSCGIOS	ScanBody Full Arch Tracer
JDSCGIOSR	ScanBody Rectangular Full Arch Tracer **
ESTIBC	TiBase Engaging Ø4.25 H 0.5 C 4.7 JDEvolution S
ESTIBNEC	TiBase Non Engaging Ø4.25 H 0.5 C 4.7 JDEvolution S
ESTIB15C	TiBase Engaging Ø4.25 H 1.5 C 4.7 JDEvolution S
ESTIB15NEC	TiBase Non Engaging Ø4.25 H 1.5 C 4.7 JDEvolution S
ESTIB30C	TiBase Engaging Ø4.25 H 3.0 C 4.7 JDEvolution S
ESTIB30NEC	TiBase Non Engaging Ø4.25 H 3.0 C 4.7 JDEvolution S
ESTIBNC	TiBase Long Engaging Ø 3.5 H 0.5 C 7.7 JDEvolution S
ESTIBNENC	TiBase Long Non Engaging Ø 3.5 H 0.5 C 7.7 JDEvolution S
ESTIB15NC	TiBase Long Engaging Ø 3.5 H 1.5 C 7.7 JDEvolution S
ESTIB15NENC	TiBase Long Non Engaging Ø 3.5 H 1.5 C 7.7 JDEvolution S
ESTIB30NC	TiBase long Engaging Ø 3.5 H 3.0 C 7.7 JDEvolution S
ESTIB30NENC	TiBase long Non Engaging Ø 3.5 H 3.0 C 7.7 JDEvolution S
ESTIBNAC	TiBase New Engaging Ø 3.5 H 0.5 C 7.7 for Angulated Channel JDEvolution S *
ESTIBNENAC	TiBase New Non Engaging Ø 3.5 H 0.5 C 7.7 for Angulated Channel JDEvolution S *
ESTIB15NAC	TiBase New Engaging Ø 3.5 H 1.5 C 7.7 for Angulated Channel JDEvolution S *
ESTIB15NENAC	TiBase New Non Engaging Ø 3.5 H 1.5 C 7.7 for Angulated Channel JDEvolution S *
ESTIB30NAC	TiBase New Engaging Ø 3.5 H 3.0 C 7.7 for Angulated Channel JDEvolution S *
ESTIB30NENAC	TiBase New Non Engaging Ø 3.5 H 3.0 C 7.7 for Angulated Channel JDEvolution S *
EVAITEC	Interface CAD CAM for Conical Abutment Engaging JDEvolution (H 7.5)
EVA139C	TiBase Engaging for Conical Abutment (H 5.0)
EVAITC	Interface CAD CAM for Conical Abutment Non Engaging JDEvolution (H 7.3)
EVCATIBC	TiBase for Conical Abutment JDEvolution (H 5.0)
EVAITAC	Interface CAD CAM Non Engaging for Conical Abutment Angulated Channel (H 7.7)
EVS110	Prosthetic Screw Conical Abutment Direct **
EVS111	Prosthetic Screw Conical Abutment Direct Long **
EVS112	Prosthetic Screw for Angulated Channel Conical Abutment Direct ***
ESANCN	Implant Replica CAD CAM New JDEvolution S
EVCAARCN	Conical Abutment Replica CAD CAM New
ESPMNC	Pre-milled Abutment Blank JDEvolution S

* for this screw use Angulated Screw Driver for JDTorque

** for screw use JDEvolution Plus prosthetic screwdriver

*** for screw use Angulated Screw Driver for Conical Abutment Direct JDTorque




Torque recommended 30 Ncm



Torque recommended 30 Ncm



Torque recommended 30 Ncm

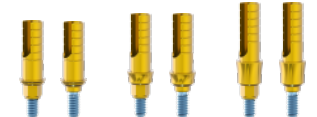

Torque recommended 15 Ncm



Torque recommended 15 Ncm



Torque recommended 30 Ncm



Implant Replica:

ESAN	Implant Replica JDEvolution S
------	-------------------------------



Ball Abutment:

ESBA15	Ball Abutment H 1.5 JDEvolution S
ESBA30	Ball Abutment H 3.0 JDEvolution S
ESBA50	Ball Abutment H 5.0 JDEvolution S
EVBA	Ball Abutment Replica
EVBAHC	Ball Abutment Elastic Retention Cap w/ Container
EVBA	Ball Abutment Retention Cap


Torque recommended 30 Ncm



Emi Abutment:


ESEMI15	Emi Abutment H 1.5 JDEvolution S
ESEMI30	Emi Abutment H 3.0 JDEvolution S
ESEMI50	Emi Abutment H 5.0 JDEvolution S
EVEMISCK	Smart cap attachment Emi Abutment
EVEMIHC	Emi Abutment Elastic Retention Cap w/ Container
EVEMIN	Emi Abutment Elastic Retention Cap (1.2 kg)
EVEMIH	Emi Abutment Container
EVEMIIC	Emi Abutment Plastic Impression Coping
EVEMIICS	Emi Abutment Steel Impression Coping
EVEMINT	Emi Abutment White Retention Cap (1.8 kg)
EVEMINY	Emi Abutment Yellow Retention Cap (0.6 kg)
EVEMINP	Emi Abutment Purple Retention Cap (2.5 kg)
EVEMINB	Emi Abutment Retention Cap for Laboratory
EVEMIEIT	Extractor-insertion Tool For Caps
EVEMIA	Emi Abutment Replica

Torque recommended 30 Ncm

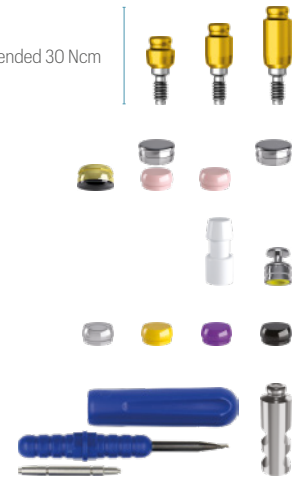

Bone Mill:

ESBM5C	Bone Mill Ø 5.0 and Guide JDEvolution S
ESBM6C	Bone Mill Ø 6.0 and Guide JDEvolution S
JDBM3	Bone Mill Guide JDEvolution S

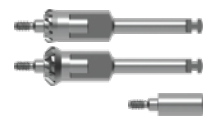

Implant and Prosthetic Drivers:

ESID	Implant Driver JDEvolution S
ESIDL	Implant Driver Long JDEvolution S
EVSDPF15	Prosthetic Driver for JDTorque L 15
EVSDPF20	Prosthetic Driver for JDTorque L 20
EVSDPF25	Prosthetic Driver for JDTorque L 25
EVSDPF35	Prosthetic Driver for JDTorque L 35
EVSDPF60	Prosthetic Driver for JDTorque L 60
EVSDCAF	Conical Abutment Driver for JDTorque
EVSDP20	Prosthetic Driver L 20
EVSDP25	Prosthetic Driver L 25
EVSDP30	Prosthetic Driver L 30
JDPD103	Prosthetic Driver for Surgical Driver
EVSDCA	Conical Abutment Driver
EVSDPF25A	Angulated Screw Driver for JDTorque L 25
EVSDPF30A	Angulated Screw Driver for JDTorque L 30
EVSDPF25AV	Angulated Screw Driver for Conical Abutment Direct JDTorque L 25
EVSDPF30AV	Angulated Screw Driver for Conical Abutment Direct JDTorque L 30
EVSDP32AV	Screw Driver for Conical Abutment Direct EVS112 (Straight Channel)
JDG	JDGuide
ESAMT	Conical Abutment Aligning Instrument JDEvolution S
ESGMC	Implant Mounter for JDEvolution S

